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Instructions for Use

Styptic Wound-Foil "NewPro Emplastrum"

**Congratulations on purchasing your
"NewPro Emplastrum".plaster**

„NewPro Emplastrum“ is made of high-quality polymer material. The material has been tested for biocompatibility and skin tolerability. „NewPro Emplastrum“ is a medicinal product without any chemical additives which is used for the compressive treatment of small, bleeding wounds.

The "NewPro Emplastrum" is intended for only one person and, after thorough cleaning and disinfection, can be used several times a day up to the final date of expiry.

Cleaning, Disinfection and Care Instructions

The "NewPro Emplastrum" has to be cleaned and disinfected as follows before the first and each further use:

1. Clean the "NewPro Emplastrum" with copious water.
2. Boil briefly. Lay the "NewPro Emplastrum" for at least 3 minutes in boiling water. Ensure that the foil can swim easily in sufficient bubbling water.
3. It is necessary that the boiling water is kept at 100°C during the whole 3 minutes sterilisation time. A shorter boiling time or lower temperature cannot guarantee a safe disinfection. Temperatures higher than 100°C or boiling for longer than three minutes does not damage the product or its effectiveness.

4. Dry the "NewPro Emplastrum" after sterilisation with a paper towel (Kitchen Roll) and store hygienically for the next use.
5. A hygienical storage is only ensured when the storage container and the Fissure Stick have been disinfected according to paragraphs 2 and 3 above. Appropriate containers are available from chemists (e.g. Pillboxes etc.)
6. If no suitable container is available according to paragraph 5, then "NewPro Emplastrum" has to be cleaned resp. disinfected again according to para. 1 to 3 before further use.

Hygienical Preparation and Practical Tips

- Press "NewPro Emplastrum" onto the bleeding wound.
- To achieve the most intensive effect maintain pressure for 2 to 4 minutes.
- Disinfect and maintain "NewPro Emplastrum" and store according to Paragraphs 1 to 4 above after use.

Special Instructions

- Do not use any disinfection agent!
- Contra indications, side effects, interactions and allergies are not known.
- The product is to be destroyed and disposed of in the event of damage to the "NewPro Emplastrum".
- Keep out of the reach of children!
- **Shelf Life:** 5 years after date of manufacture if used correctly.

Why does "NewPro Emplastrum" help?

- "NewPro Emplastrum" is made of an inactive plastic with excellent bio-compatibility. It is not attacked by solutions nor by other aggressive chemicals. It shows no changes at all after contact with the most chemicals. Its surface is so smooth and slippery that hardly any foreign substance can stick to it. So it cannot stick to the wound area. These properties accelerate the wound healing, when "NewPro Emplastrum" lies tightly pressed on the wound. Through the pressure and the fine pores of the surface of "NewPro Emplastrum", the leaking of wound liquids or blood serum -which every wound secretes – is prevented.
- A skin is formed very quickly within this blood serum, as for example in the case of a foot blister, and the wound heals.

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Closes wounds in seconds

No burning pain, no wound liquids, no scab forming

Can always be reused

Intended for one person only

Distinguished with the ihoch-Innovations Prize of the
Federal Ministry for Education and Research